

## Other Available *Life/form*® Simulators

- LF00698U** Adult Injectable Arm (White)
- LF00855U** Male Catheterization
- LF00856U** Female Catheterization
- LF00901U** Prostate Examination
- LF00906U** Ostomy Care
- LF00929U** Surgical Bandaging
- LF00957U** Enema Administration
- LF00958U** Pediatric Injectable Arm
- LF00961U** Intramuscular Injection
- LF00984U** Breast Examination
- LF00995U** Arterial Puncture Arm
- LF00999U** Pediatric Injectable Head
- LF01005U** First Aid Arm
- LF01008U** Intradermal Injection Arm
- LF01012U** Heart Catheterization (TPN)
- LF01019U** Ear Examination
- LF01027U** Peritoneal Dialysis
- LF01028U** Suture Practice Arm
- LF01034U** Suture Practice Leg
- LF01036U** Spinal Injection
- LF01037U** Hemodialysis Practice Arm
- LF01038U** Episiotomy Suturing Set
- LF01042U** Suture Kit
- LF01062U** Pelvic, Normal & Abnormal
- LF01063U** Stump Bandaging, Upper
- LF01064U** Stump Bandaging, Lower
- LF01069U** Cervical Effacement
- LF01070U** Birthing Station
- LF01082U** Cricothyrotomy
- LF01083U** Tracheostomy Care
- LF01084U** Sigmoidoscopic Examination
- LF01087U** Central Venous Cannulation
- LF01095U** Blood Pressure Arm
- LF01108U** Infant Intraosseous Infusion
- LF01121U** Advanced IV Arm
- LF01131U** Venipuncture and Injection Arm
- LF01139U** Advanced IV Hand
- LF01142U** Auscultation Trainer
- LF01143U** Testicular Exam
- LF01152U** Male & Female Catheter
- LF01155U** Advanced CPR Dog
- LF01162U** Venatech IV Trainer
- LF01174U** NG Tube & Trach Skills
- LF01184U** Venatech IM & Sub Q
- LF01193U** Special Needs Baby
- LF03000U** **CPARLENE**® Series
- LF03601U** Adult Airway Management Trainer with Stand
- LF03602U** Adult Airway Management Manikin
- LF03609U** Child Airway Management Trainer with Stand
- LF03616U** Child **CRiSis**™ Manikin
- LF03617U** Deluxe Child **CRiSis**™ Manikin with Arrhythmia Tutor
- LF03620U** PALS Update Kit
- LF03623U** Infant Airway Management Trainer with stand
- LF03632U** Child Intraosseous Infusion/ Femoral Access Leg on a Stand
- LF03633U** Child Airway Management Trainer Torso
- LF03693U** **Basic Buddy** CPR Manikin
- LF03699U** "Airway Larry" Airway Management Trainer
- LF03709U** Infant **CRiSis**™ Manikin
- LF03720U** **Baby Buddy** Infant CPR Manikin
- LF03750U** Fat Old Fred
- LF03760U** Airway Management/Cricoid Pressure Trainer
- LF03770U** Chest Tube
- LF03953U** **CRiSis**™ Manikin, complete
- LF03955U** Deluxe **CRiSis**™ Manikin
- LF03956U** Deluxe "Plus" **CRiSis**™ Manikin
- LF03965U** Adult **CRiSis**™ Auscultation Manikin
- LF03966U** Adult **CRiSis**™ Auscultation Manikin with ECG Simulator
- LF04000U** **GERI**™ / **KERI**™ Manikin Series
- LF04200U** Adult Sternal Intraosseous Infusion
- LF06001U** **CPR Prompt**™ Adult/Child Manikin
- LF06012U** **CPR Prompt**™ Infant Manikin
- LF06200U** **CPR Prompt**™ Keychain Rescue Aid
- LF06204U** **CPR Prompt**™ Rescue and Practice Aid

*Nasco* Fort Atkinson  
901 Janesville Avenue, P.O. Box 901  
Fort Atkinson, Wisconsin 53538-0901  
1-800-558-9595  
www.eNasco.com • E-mail: lifeform@eNasco.com

COPYRIGHT © NASCO 2008 PRINTED IN U.S.A. NP 117-08

*Nasco*  
**Life/form**®



# BASIC **CRiSis**™ CHILD MANIKIN SIMULATOR LF03765U INSTRUCTION MANUAL



**WARNING:** Products may contain dry natural rubber.

*Life/form*® Products by Nasco





**UPDATE YOUR BASIC *Life/form*® CHILD *CRiSis*™ MANIKIN WITH *Life/form*® MODULAR COMPONENTS:**

- LF03611U** Defibrillation Chest Skin
- LF03612U** Injectable Training Arm
- LF03613U** Blood Pressure Arm
- LF03614U** Intraosseous Infusion/Femoral Access Leg
- LF03618U** Intraosseous Replacement Bones
- LF03626U** Femoral Replacement Injection Pad
- LF03619U** Intraosseous Replacement Skin
- LF03683U** Intraosseous Infusion Needle
- LF03628U** Replacement Stomach
- LF03627U** Replacement Lung Set Only
- LF03670U** Interactive ECG Simulator

<b>Table of Contents</b> . . . . .	<b>Page</b>
<b>Introduction</b> . . . . .	<b>2</b>
<b>Assembly of your Basic Child <i>CRiSis</i>™ Manikin</b> . . . . .	<b>2</b>
<b>About the Basic <i>Life/form</i>® Child Airway Management Trainer Head</b> . . . . .	<b>3</b>
<b>Additional Supplies to Upgrade the Manikin.</b> . . . . .	<b>4</b>



Figure 1

### Basic **Life/form**® Child **CRiSis**™ Manikin

This full-body child manikin has been created to offer realistic training and practice of child airway management and CPR education. The Basic **Life/form**® Child **CRiSis**™ Manikin is anatomically correct in detail and represents the size of an 8-year-old child. Realistic anatomy and landmarks include a mouth, tongue, oral pharynx, larynx, epiglottis, arytenoids, vocal cords, trachea, and esophagus. In addition to the palpable and visual landmarks, the head, neck, and jaw are fully articulated, allowing a realistic jaw thrust feature. This manikin has separate left and right lungs for auscultation; has oral, nasal, and digital intubation capabilities; and includes an inflatable stomach bladder to indicate esophageal intubation. Also features E.T., E.O.A., P.T.L., L.M.A., E.G.T.A., and Combitube® Insertion. This basic airway management trainer may be gradually updated to the **Life/form**® Child **CRiSis**™ or Deluxe Child **CRiSis**™ manikins by adding the available upgrades and the Interactive ECG Simulator (LF03670U). Includes pump spray lubricant. See page 4 for complete list of modular update components. Three-year warranty.



Figure 2

### Assembly of your Basic Child **CRiSis**™ Manikin:

To facilitate packing, your manikin comes with the arms and legs unattached.

To connect the arms to the torso, place each in a position such that the hand is above the head. (See figure 1.)

Align the arm fitting with the torso socket and plug the arm into place. Finish by rotating the arm forward so it lies along the side of the body.

To connect the legs to the torso, simply slide the waist pin (supplied) through the torso and legs, pinning the legs in their correct position. (See figure 2.)

## THE AIRWAY MANAGEMENT TRAINER



### About the Simulator...

The Basic **Life/form**® Child Airway Management Trainer Head is the most realistic simulator available for the training of intubation and resuscitation skills. Nasco has taken great care to create a trainer that is anatomically correct in respect to both size and detail.

Landmarks include: nostrils, teeth, tongue, oral and nasal pharynx, larynx, epiglottis, arytenoids, false and true cords, cricoid ring, tracheal rings, trachea, esophagus, lungs, and stomach.

The Basic **Life/form**® Child Airway Management Trainer Head allows you to practice oral, digital, and nasal intubation, as well as CPR. Suction techniques can also be performed and evaluated. The simulator was designed to use an uncuffed endotracheal tube measuring up to 5.0 mm I.D.

### List of Components:

1. Child Airway Management Trainer Head
2. Right and Left Lung Simulators
3. Stomach Simulator
4. Pump Spray Lubricant
5. Three Nylon Hose Clamps



Figure 3

### Lubrication:

Before using the Basic **Life/form**® Child Airway Management Trainer, lubricate both the oral cavity and the endotracheal tube with the Nasco lubricant provided. (See figure 3.) **NOTE:** Nasco recommends the use of the provided lubricant or a similar vegetable-based lubricant for the Child Trainer. The use of a silicone or similar lubricant may seriously damage the simulator, thus voiding Nasco's warranty on the trainer.

### Cleaning the Face:

Normal soil can be removed from the face with mild soapy water. REN cleaner (W09919U) will remove stubborn stains. Simply apply REN to soiled area and wipe clean with a soft cloth or paper towel. **NOTE:** Avoid using cleaner around mouth area if students will be applying direct mouth-to-mouth resuscitation techniques, as the cleaner may be toxic if ingested. NEVER place the trainer on any kind of printed paper or plastic. These materials, as well as ball-point pens, will transfer indelible stains. Do not use any cosmetics.

### AVAILABLE SUPPLIES:

- LF03631U** Replacement Headskin with Airway
- LF03627U** Replacement Lungs
- LF03628U** Replacement Stomach
- LF03644U** Pump Spray Lubricant
- W09919U** REN Cleaner